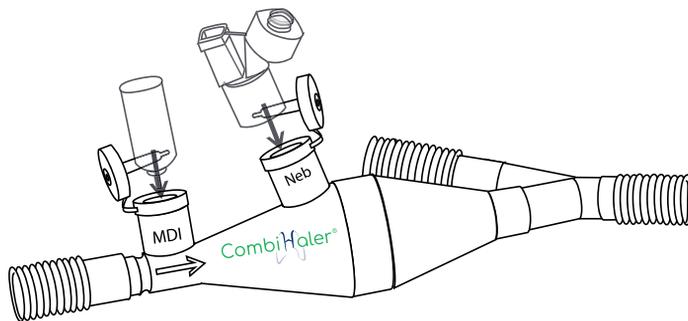
# CombiHaler®

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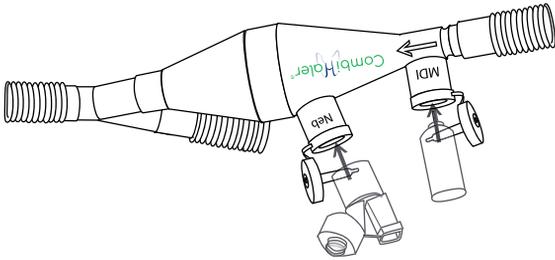
# CombiHaler®

CombiHaler® is manufactured in France by :

**Laboratoire ProtecSom Optimal SAS, ZA d'Armanville BP46, 50700 Valognes**  
Tél : +33 (0) 233 887 094 / Fax : +33 (0) 233 887 095


@ : info@optimal.com  
<http://www.optimal.com>

The trademark CombiHaler® belongs to Laboratoire ProtecSom Optimal SAS.

Patents belong to Tours University, INSERM and Laboratoire ProtecSom Optimal  
together.

CE  
0344

#### 4) Duration of use:

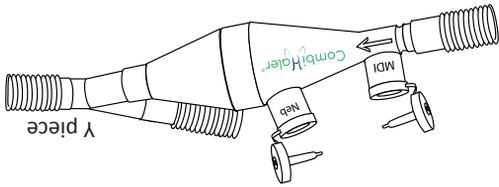Ventilator  
connexion

a. Insert the CombiHaler® into the mechanical ventilation circuit between the inspiratory limb and the «Y-piece», following the direction of the arrow as indicated on the device CombiHaler®. The MDI canister attaches to the port annotated MDI. For proper operation of the MDI, refer to the manufacturer's instruction. The nebulizer screen attaches to the port annotated Neb. For proper operation of the mesh nebulizer, refer to the manufacturer's instructions.

b. Using an MDI with CombiHaler® :

- i. Shake the MDI canister according to the manufacturer's instructions.
- ii. Open the sealing cap of the MDI port and insert the canister, valve stem facing down.

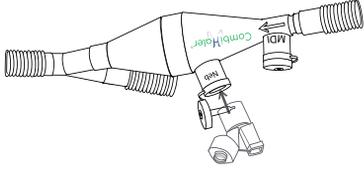
- iii. Actuate the MDI at the beginning of the respiratory cycle.
- iv. Let the inspiration from the ventilator administer the drug dose to the patient. Remove the MDI canister and place the cap back in place.
- v. Repeat steps «i» to «v» depending on the number of doses needed.


c. Using a vibrating mesh nebulizer with CombiHaler® :

- i. Insert the nebulizer into the Neb port.
- ii. Place the drug solution in the reservoir of the nebulizer.

iii. For proper operation of the nebulizer, follow the instructions provided by the manufacturer.

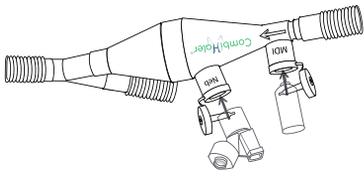d. CombiHaler® can be used simultaneously with both an MDI and a vibrating mesh nebulizer.


# Manual of CombiHaler™ to mechanically ventilated patients

## 1) Description :

CombiHaler™ is a spacer designed for invasive and noninvasive mechanical ventilation systems. It allows for the simultaneous use of a pressurized metered dose inhaler and a vibrating mesh nebulizer.

It consists of :

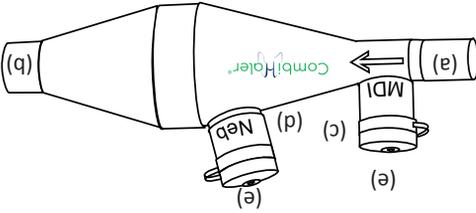- a. An entry
- b. An output
- c. An opening for MDI
- d. An opening for Nebulizer
- e. Two sealing caps

## 2) Special precautions:

a. CombiHaler™ ships fully assembled. It should never be opened before use. Any CombiHaler™ which has been opened should be discarded and replaced.

b. When commissioning, it is necessary to ensure that :

- i. The device does not contain any foreign body.
- ii. The caps are positioned correctly when no MDI and / or nebulizer are used.
- iii. The MDI nozzle functions properly (tested visually with a puff of product.)

c. During use it is necessary to ensure that :

- i. The caps are positioned correctly when no MDI and / or nebulizer are used.
- ii. The MDI nozzle functions properly (visual inspection during each puff of product release.)

## 3) Duration of use:

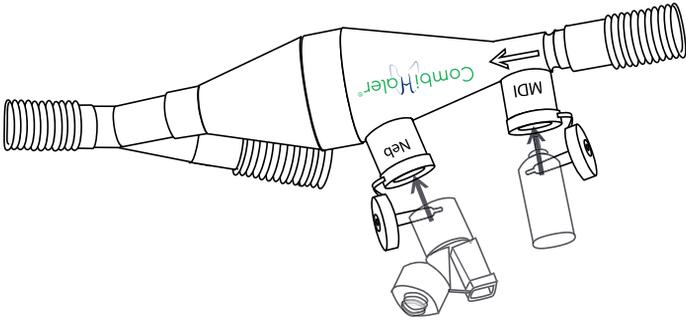
a. CombiHaler™ is a non-sterile device for single patient use.

b. CombiHaler™ can be used throughout the life of the ventilating circuit for no more than seven consecutive days

c. CombiHaler™ should never be used on more than one circuit and must be destroyed under the same conditions as the circuit on which it was mounted.

d. Any total or partial replacement of the circuit should result in the replacement of CombiHaler™.

CombiHaler®

Operating manual